


# Plaque Test

Fluorescent disclosing liquid

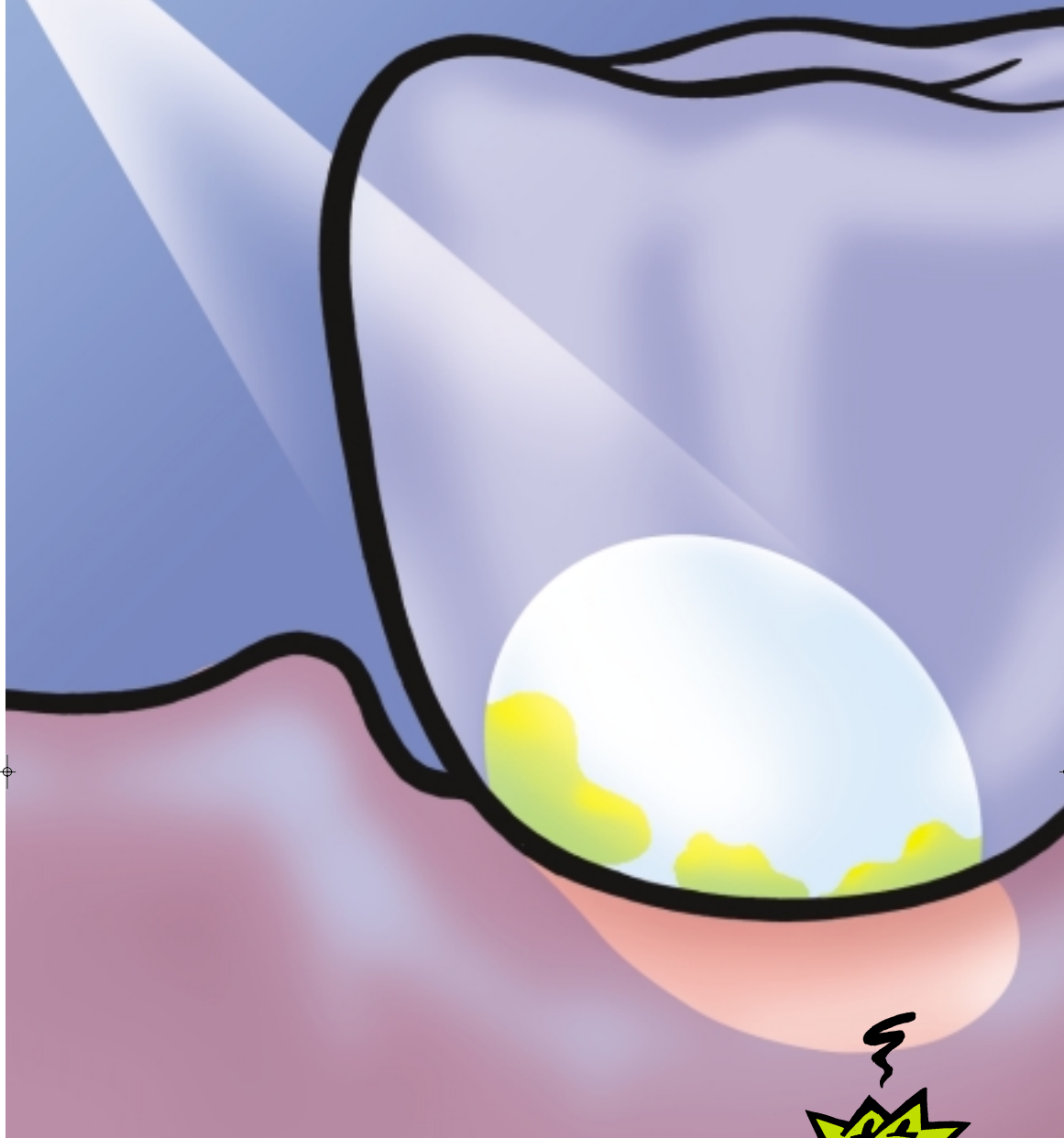

Makes plaque visible

# Plaque Test

The convincing solution for the exposure of plaque

## Visible

With the polymerization light



## Invisible

Without the polymerization light



Bacterial plaque promotes the development not only of dental caries, but also of periodontal diseases, gravely endangering the health of teeth and gums.

Plaque Test is a fluorescent disclosing liquid for the exposure of plaque.

## Illuminating the Plaque

- ➔ Apply Plaque Test generously to the surfaces of the teeth
- ➔ Ask the patient to rinse
- ➔ Illuminate the teeth with a polymerisation lamp. Any areas affected by plaque appear brightly fluorescent.
- ➔ Thoroughly remove the now visible plaque via tooth brushing or professional tooth cleaning.

## Advantages:

- ➔ fluoresces under every curing light
- ➔ no discoloration of tongue or mucous membranes.
- ➔ easy handling
- ➔ targeted application
- ➔ economical use

## The convincing solution!

Plaque Test is the quick, easy and aesthetic way to:

### Convince

your patients of the need for better oral hygiene measures

### Instruct

your patients where and how to perform these measures

### Protect

your patients from avoidable dental and periodontal diseases

## Delivery form

1 Bottle  
Plaque Test, 10 ml  
Accessories



## Production and Distribution

Ivoclar Vivadent AG  
Bendererstrasse 2  
FL-9494 Schaan  
Principality of Liechtenstein  
T +423 / 235 35 35  
F +423 / 235 33 60  
www.ivoclarvivadent.com

