Te-Econom Plus® Te-Econom Flow® Te-Econom Bond® Eco-Etch®

Návod na použitie
Használati utasítások
Instrukcja stosowania
Упутство за употребу
Упатство за употреба
Указания за употреба
Udhëzim për përdorim
Instruqioni de utilizare
Kullanım Kılavuzu

Instructions for Use
Mode d'emploi
Instrucciones de uso
Instruções de Uso
Инструкция
Navodilo za uporabo
Uput za uporabu
Návod k použití

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Instructions for Use

Descriptions

Eco-Etch is a phosphoric acid gel for enamel etching and dentin conditioning.

Te-Econom Bond is a single-component bonding agent for enamel and dentin bonding in conjunction with the total-etch technique.

Te-Econom Flow is a flowable, light-curing, radiopaque hybrid composite.

Te-Econom Plus is a light-curing, radiopaque composite for restorative treatment.

Te-Econom Plus, Te-Econom Flow and Te-Econom Bond cure with light in the wavelength range of 400–500 nm (blue light).

Shades

Te-Econom Plus is available in shades A1, A2, A3, A3.5, B2, C3

Te-Econom Flow is available in shades A2 and A3

Compositions

Eco-Etch contains phosphoric acid (37 % wt in water), thickeners and pigments.

Te-Econom Bond consists of HEMA, di- and monomethacrylates, inorganic fillers, initiators and stabilizers in an alcohol solution.

Te-Econom Flow consists of dimethacrylates (37 % wt), inorganic fillers (62 % wt or 38 % vol) with a particle size between 0.04 and 7 µm, initiators, stabilizers and pigments (1 % wt).

Te-Econom Plus consists of dimethacrylate and TEGDMA (22 wt%). The fillers include barium glass, ytterbium trifluoride, silicon dioxide and mixed oxide (76 wt% or 60 vol%). Additives, initiators, stabilizers and pigments are additional contents (2 wt%). The particle size of inorganic fillers is between 0.04 and 7 µm. The mean particle size is 850 nm.

Indication

Eco-Etch is used in the enamel etch or total-etch technique in conjunction with adhesives, luting materials and fissure sealants.

Te-Econom Bond/Te-Econom Flow/Te-Econom Plus

- Restorations of Class I to V cavities (small cavities of Class I and II for Te-Econom Flow)
- Restoration of deciduous teeth

Contraindication

- If a dry working field cannot be established or the stipulated application technique cannot be applied.

- If a patient is known to be allergic to any of the ingredients.
- Te-Econom Bond must not be used in conjunction with self-curing composite materials.
- Direct pulp capping with Te-Econom Bond/Flow/Plus.

Side effects

- In rare cases, ingredients of Te-Econom Bond/Flow/Plus may lead to sensitization. The products should not be used in such cases.
- To avoid possible irritation of the pulp, areas close to the pulp should be protected with a suitable pulp/dentin protector.

Interaction

- Glass ionomer-based liners may be dissolved by Eco-Etch.
- Substances containing eugenol/clove oil inhibit the polymerization of resin-based materials.
- Discolouration may occur in combination with cationic mouthwashes, plaque disclosing agents and chlorhexidine.

Application

1. Select the shade.
For facilitated access to the treatment field, the flexible and patient-friendly mouth opener OptraGate® can be used.
2. Establish sufficient isolation, preferably with a rubber dam (complete isolation is ensured with, for example, the anatomically shaped OptraDam® Plus).
3. Prepare the cavity according to the requirements of the adhesive technique.
4. Clean the cavity with water spray.
5. Air dry the cavity.
6. Apply a pulp protector (calcium hydroxide) if needed; only cover areas close to the pulp and subsequently apply a pressure-resistant cement (e.g. Vivaglass® Liner).
7. Place a matrix (e.g. the selectively thinned OptraMatrix®) and an interdental wedge, if required.
8. Apply Eco-Etch on the enamel and then on dentin and allow a reaction time of 30 seconds on enamel and 15 seconds on dentin. After that, thoroughly rinse with water spray and dry the tooth surfaces with oil-free compressed air. Avoid dehydrating the dentin.
9. Apply a generous amount of Te-Econom Bond on enamel and dentin, using the enclosed applicator brush. Brush the material gently into the dentin for 10 s. Next, evaporate any remaining solvent with a slow stream of air. Lightcure the adhesive according to the curing time recommendations below. Ensure complete coverage of the preparation surface.
10. Apply Te-Econom Flow in layers of a max. of 2 mm and light cure according to the curing recommendations below.
Apply Te-Econom Plus in layers of max. 2 mm and adapt/contour the material with a suitable instrument (e.g. OptraSculpt®). The

OptraContact® instrument can be used to selectively position high-quality proximal contacts. Polymerize each layer individually according to the table below. Hold the light emission window as closely as possible to the surface of the restorative material.

Light intensity	Material	Turbo Programm	High Power Programm	Soft Start Programm	Low Power Programm
LEDition (600 mW/cm ²)	Te-Econom Bond	–	10 s	–	–
	Te-Econom Flow	–	20 s	–	–
	Te-Econom Plus	–	20 s	–	–
bluephase C8 (800 mW/cm ²)	Te-Econom Bond	–	–	–	10 s
	Te-Econom Flow	–	15 s	20 s	–
	Te-Econom Plus	–	15 s	20 s	–
bluephase (1.200 mW/cm ²)	Te-Econom Bond	–	–	–	10 s
	Te-Econom Flow	–	10 s	15 s	–
	Te-Econom Plus	–	10 s	15 s	–
bluephase 20i (2.000 mW/cm ²)	Te-Econom Bond	–	–	–	10 s
	Te-Econom Flow	5 s	10 s	15 s	–
	Te-Econom Plus	5 s	10 s	15 s	–

Light intensity	Material	Te-Econom Plus	Te-Econom Flow
	≥ 500 mW/cm ²		20 s
≥ 1.000 mW/cm ²		10 s	10 s

11. Finish the restoration with suitable finishers or fine diamonds. Subsequently, check the occlusion. Polish with silicone polishers (e.g. OptraPol® Next Generation).

Special note

If fresh composite is applied to polished material, it must first be roughened and wetted with Heliobond.

Warning

- Eco-Etch contains phosphoric acid and is corrosive. Avoid contact with the eyes, mucous membranes, and skin. Goggles are recommended for both patients and operators.
- Unpolymerized Te-Econom Bond/Flow/Plus should not come in contact with skin/mucous membranes. The unpolymerized products may have an irritating effect and lead to sensitization against methacrylates.
- After contact of Eco-Etch or Te-Econom Bond with the eyes, immediately rinse with copious amounts of water for at least 15 minutes and see a physician/ophthalmologist.

- After skin contact with Eco-Etch or Te-Econom Bond, immediately rinse with soap and water. After contact with sensitive materials (surfaces, clothing), immediately clean with copious amounts of water.
- Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

Storage and hygiene

- Storage temperature 2–28 °C (36–82 °F).
- Close syringe and bottle immediately after use. Exposure to light causes premature polymerization.
- Expiry date: see note on respective delivery form.
- Do not use any of these products after the indicated date of expiration.
- Do not disinfect syringes with oxidizing disinfectants.
-  If Te-Econom Plus is applied from the Cavifil directly in the mouth of the patient, the Cavifil must not be used for more than one patient due to hygienic reasons (prevention of cross-contamination between patients).

Keep out of the reach of children! For use in dentistry only!

Date information prepared: 06/2010; Rev. 2

Manufacturer

Ivoclar Vivadent AG
FL-9494 Schaan/Liechtenstein

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes.

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Jačina svjetlosti	Materijal	Turbo program	Program High Power	Program Soft Start	Program Low Power
LEDition (600 mW/cm ²)	Te-Econom Bond	–	10 s	–	–
	Te-Econom Flow	–	20 s	–	–
	Te-Econom Plus	–	20 s	–	–
bluephase C8 (800 mW/cm ²)	Te-Econom Bond	–	–	–	10 s
	Te-Econom Flow	–	15 s	20 s	–
	Te-Econom Plus	–	15 s	20 s	–
bluephase (1.200 mW/cm ²)	Te-Econom Bond	–	–	–	10 s
	Te-Econom Flow	–	10 s	15 s	–
	Te-Econom Plus	–	10 s	15 s	–
bluephase 20i (2.000 mW/cm ²)	Te-Econom Bond	–	–	–	10 s
	Te-Econom Flow	5 s	10 s	15 s	–
	Te-Econom Plus	5 s	10 s	15 s	–

Jačina svjetlosti	Materijal	Te-Econom Plus	Te-Econom Flow
	≥ 500 mW/cm ²		20 s
≥ 1.000 mW/cm ²		10 s	10 s

11. Izrada restoracije pomoću prikladnih finirajućih instrumenata ili sitnozrnastih dijamanta i konačnom okluzivnom kontrolom. Poliranje pomoću silikonskih finirajućih instrumenata (npr. OpraPol® Next Generation).

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Ivoclar Vivadent UK Limited
Ivoclar Vivadent, Inc.