Description

Systemp®.desensitizer is an aqueous solution for reducing dentinal hypersensitivity and postoperative sensitivity. Systemp.desensitizer is suitable for the following fields of application: temporary restorations, luting/cementation of prosthetic restorations, direct restorative treatment and treatment of sensitive tooth necks. Systemp.desensitizer is a clear solution for application onto dentin surfaces.

Composition

Poly(ethylene glycol)dimethacrylate and glutaraldehyde in an aqueous solution.

Indications

Prevention and reduction of dentin hypersensitivity and postoperative sensitivity in conjunction with:

- Temporary restorations
- Luting/Cementation of indirect restorations
- Direct restorative therapy
- Sensitive tooth necks

Contraindication

- If a patient is known to be allergic or hypersensitive to any of the material's ingredients.
- If the pain is caused by pulpitis or inflammation.

Side effects

In rare cases, ingredients of Systemp.desensitizer may cause a sensitizing reaction in patients with a hypersensitivity to any of the material's ingredients. In these cases, the material should no longer be used.

Interactions

Interaction with other materials (adhesives, cements, temporaries, restoratives) applied to the tooth structure may occur. These interactions may, for example, affect the bond strength of adhesives or cements.

Application**1. Temporary restoration**

- Prepare cavity / abutment.
- You may take the impression at this stage or wait until you have applied the desensitizer (see notes).
- Dentin surfaces must be clean and dry. Keep the working field dry (e.g. cotton rolls).
- Apply Systemp.desensitizer and gently brush it into the dentin for 10 seconds using a suitable instrument (brush, applicator brush).
- Carefully disperse excess with blown air / dry with an air syringe. Do not overdry the dentin.
- Fabricate the temporary restoration and provisionally cement the restoration.

2. Luting / Cementation

- Remove the temporary restoration.
- Clean the preparation, e.g. with a brush, rubber cup, and cleaning paste.
- Dentin surfaces must be clean and dry. Keep the working field dry (e.g. cotton rolls).
- Apply Systemp.desensitizer and gently brush the material into the dentin for 10 seconds using a suitable instrument (brush, applicator brush).
- Carefully disperse excess with blown air / dry with an air syringe. Do not overdry the dentin.
- Try in the restoration.
- Seat the restoration.

3. Direct restorations

- Prepare cavity, excavate caries.
- Dentin surfaces must be clean and dry. Keep the working field dry (rubber dam, e.g. OptraDam® Plus).
- Apply Systemp.desensitizer and gently brush the material into the dentin for 10 seconds using a suitable instrument (brush, applicator brush).
- Carefully disperse excess with blown air / dry with an air syringe. Do not overdry the dentin.
- Apply a base / liner, if necessary.
- Place the restoration.

Notes

- The application of Systemp.desensitizer slightly increases the adhesion of Systemp.inlay and Systemp.onlay to the cavity walls. Nevertheless, they can be removed at the second appointment without difficulty.
- If light-curing temporary C&B materials are used, Systemp.desensitizer should be applied after the fabrication of the temporary. If you apply the desensitizer prior to fabricating the temporary, the C&B material may react with Systemp.desensitizer. As a result the material may stick to the tooth structure and may be difficult to remove.
- The application of Systemp.desensitizer during temporary fabrication does not affect or limit the choice of the cementation system or the success of the cementation procedure in the second appointment. Both conventional and adhesive cementation systems can be used for cementation purposes.
- Coating of enamel with Systemp.desensitizer does not create any problems and does not affect the subsequent treatment steps and materials being used. (Thoroughly remove excess desensitizer / dry.)
- Systemp.desensitizer can be applied before or after impression-taking. The layer thickness is very thin (the accuracy of fit is maintained, even if the desensitizer is applied after impression-taking). Interaction of the desensitizer with impression materials are not known (if applied before impression-taking).

Warnings

Systemp.desensitizer contains glutaraldehyde and is detrimental to health if inhaled or swallowed. Avoid contact with the skin, mucous membrane and eyes. If the material comes into contact with the eyes, immediately rinse with copious amounts of water and seek medical advice. Repeated skin contact and inhalation may cause sensitization. Commercially available medical gloves do not offer protection against the sensitizing effect of methacrylates.

Systemp[®].desensitizer

EN	Instructions for Use Desensitizer agent	PT	Instruções de Uso Agente dessensibilizante
FR	Mode d'emploi Désensibilisant dentinaire	TR	Kullanma Talimatı Hassasiyet giderici
ES	Instrucciones de uso Desensibilizante	RU	Инструкция по применению десенсилайзер

Storage

- Shelf life: see date of expiration on label/packaging
- Do not use Systemp.desensitizer after the indicated expiration date.
- Close bottle immediately after use.
- Storage temperature: 2 – 28°C / 36 – 82°F.

Keep out of the reach of children!

For use in dentistry only!

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions.

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Ivoclar Vivadent Marketing Ltd.
Ivoclar Vivadent AB
Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.